

CoughAssist T70

Caregiver competency checklist

Patient name: _____ Caregiver name: _____

| Goals | Date goals met | Trainer |
|--|----------------|---------|
| The caregiver will be able to: | | |
| 1. Understand basic indications for CoughAssist T70 usage | | |
| 2. Explain the buttons, indicators, and/or connectors on the front, rear, and side panels of the device Front panel <input type="checkbox"/> Start/stop button <input type="checkbox"/> Left button <input type="checkbox"/> Right button <input type="checkbox"/> Center button <input type="checkbox"/> Manual switch Rear and side panels <input type="checkbox"/> AC power inlet <input type="checkbox"/> Fan exhaust <input type="checkbox"/> Remote control connector (if applicable) <input type="checkbox"/> SpO ₂ connector (if applicable) <input type="checkbox"/> DC power inlet (if applicable) <input type="checkbox"/> Filter <input type="checkbox"/> Detachable battery slot (if applicable) <input type="checkbox"/> Tubing retainer <input type="checkbox"/> SD card slot <input type="checkbox"/> Breathing circuit connection | | |
| 3. Connect to AC power | | |
| 4. Assemble and connect the <input type="checkbox"/> Patient interface <input type="checkbox"/> Tubing <input type="checkbox"/> Bacteria filter | | |

| Goals | Date goals met | Trainer | | |
|---|---|--|--|--|
| Patient/caregiver will be able to: | | | | |
| 5. Change therapy settings in the Setting access screen (if applicable) <ul style="list-style-type: none"> <input type="checkbox"/> Preset <input type="checkbox"/> Mode <input type="checkbox"/> Cough-Trak <input type="checkbox"/> Inspiratory pressure <input type="checkbox"/> Expiratory pressure <input type="checkbox"/> Inspiratory time <input type="checkbox"/> Expiratory time <input type="checkbox"/> Pause time <input type="checkbox"/> Flow | | | | |
| 6. Connect to DC power and explain power source indicators <ul style="list-style-type: none"> <input type="checkbox"/> External battery <input type="checkbox"/> Detachable battery pack | | | | |
| 7. Understand the information on the monitoring screen <table border="0" style="width: 100%; margin-top: 10px;"> <tr> <td style="width: 50%; vertical-align: top;"> Detailed view off <ul style="list-style-type: none"> <input type="checkbox"/> Status panel <input type="checkbox"/> Manometer panel <input type="checkbox"/> Parameters panel <input type="checkbox"/> Soft keys </td> <td style="width: 50%; vertical-align: top;"> Detailed view on <ul style="list-style-type: none"> <input type="checkbox"/> Status panel <input type="checkbox"/> Manometer panel <input type="checkbox"/> Parameters panel <input type="checkbox"/> Soft keys </td> </tr> </table> | Detailed view off <ul style="list-style-type: none"> <input type="checkbox"/> Status panel <input type="checkbox"/> Manometer panel <input type="checkbox"/> Parameters panel <input type="checkbox"/> Soft keys | Detailed view on <ul style="list-style-type: none"> <input type="checkbox"/> Status panel <input type="checkbox"/> Manometer panel <input type="checkbox"/> Parameters panel <input type="checkbox"/> Soft keys | | |
| Detailed view off <ul style="list-style-type: none"> <input type="checkbox"/> Status panel <input type="checkbox"/> Manometer panel <input type="checkbox"/> Parameters panel <input type="checkbox"/> Soft keys | Detailed view on <ul style="list-style-type: none"> <input type="checkbox"/> Status panel <input type="checkbox"/> Manometer panel <input type="checkbox"/> Parameters panel <input type="checkbox"/> Soft keys | | | |
| 8. Describe a typical treatment | | | | |
| 9. Demonstrate how to turn therapy on | | | | |
| 10. Review care, cleaning, and maintenance | | | | |

Caregiver/patient signature

Trainer signature

