

THE BREATHE NIOV™ VENTILATION SYSTEM

Quick Start


For step-by-step instructions on using the Breathe NIOV ventilation system, see the manual *Breathe NIOV Ventilation System Patient Instructions for Use*, shipped with the ventilation system.



- 1 Oxygen cylinder
- 2 Ventilator
- 3 Belt clip
- 4 Battery charger
- 5 Interface
- 6 Oxygen supply hose
- 7 Oxygen regulator



Slide the regulator over the neck of the cylinder, and line up the pins on the regulator with the holes in the cylinder neck.

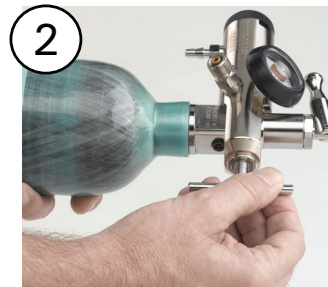

Tighten the tee screw on the regulator by turning the handle clockwise.



Push the small oxygen supply hose connector onto the ventilator oxygen supply connection until it snaps into place.

www.BreatheNIOV.com  
 United States: 1-877-698-1326  
 Technical Support and Customer Service  
 www.breathetechologies.com  
 info@breathetechologies.com  
 1-877-698-1325  
 Irvine, CA 92618 USA  
 175 Technology Drive, Suite 100  
 Breathe Technologies, Inc.




Connect the green oxygen supply hose connector to the oxygen regulator by turning it clockwise.



Plug interface into ventilator port.



Loop interface tubing over the ears so the pillows are positioned snugly inside the nostrils.

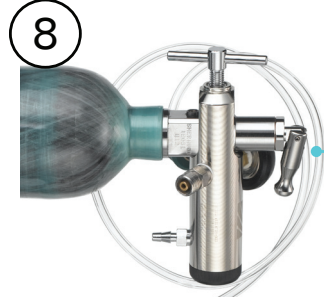


Open the cylinder air flow valve.



Adjust tubing length under the chin.



Turn on the ventilator.

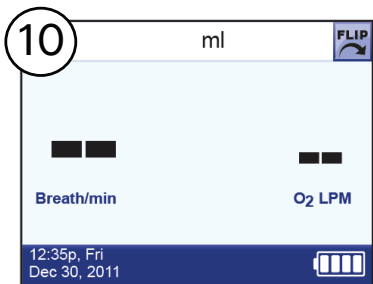

Wait until you see this screen.

### Charging the Battery




**1** Connect the battery charger cord to the ventilator charger port.

**2** Plug the power cord of the battery charger assembly into the battery charger.

**3** Plug in the wall plug.

**Note:** If the ventilator is off, the battery charger light turns green when the battery is fully charged. The ventilator can be on and in use while the battery is charging, but the battery charger light will remain red.

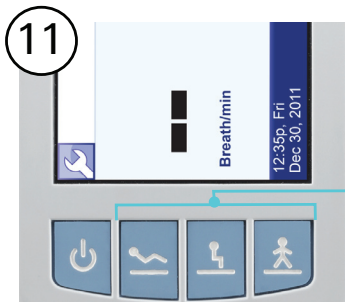

Press an **Activity** button for approximately 1 second or until you hear a tone.



Your breath rate and the oxygen flow are displayed.

Determine if the battery charge level is sufficient for your current needs. If not, charge the battery.

Charge Icon	Approximate Charge	Approximate Time
	80-100%	3 – 4 hours
	57-79%	2.5 – 3 hours
	36-56%	1.5 – 2.5 hours
	15-35%	0.5 – 1.5 hours
	< 15%	Less than 0.5 hour. Recharge immediately.
	Critically low <5%	Critically low. Recharge immediately.